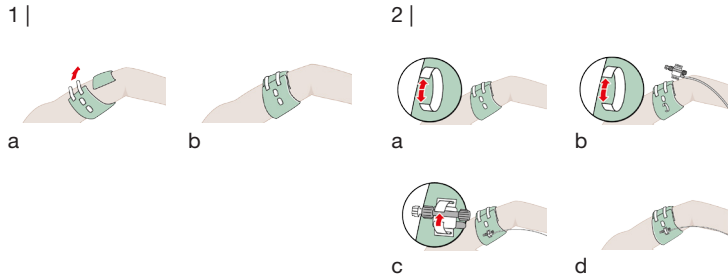
Transducer-Fix

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en

1. Product Description/Application Area

Transducer-Fix serves as a fixation for up to two transducers of invasive blood pressure measuring systems close to the body. This fixation can reduce measurement errors of hemodynamic parameters. The invasive blood pressure measurement can be continued without malfunction when it comes to repositioning or mobilization. Transducer-Fix can be used for adults and adolescents.

2. Warning Notices

When using fixations circularly, please make always sure that the application is not too tight. A sufficient blood circulation in the particular extremity must be ensured all the time. The transducer is to be positioned in a way that no pressure marks (decubitus ulcer) can occur. Any kind of strangulation should be avoided, possibly by using additional fixations for tubing systems. In case of allergic or cutaneous reactions, please remove Transducer-Fix immediately.

3. Application

- 1 a-b | Place the green cuff around the patient's arm or leg and adjust it. The white side is directed at the skin. If necessary, cut the cuff off and close it with the two hook and loop fasteners.
- 2 a-d | For fixing the transducer open one of the white hook and loop straps, put the ends through the lateral slot of the plate and close the strap thereafter. In case you use transducers without lateral slots at the plate, put the transducer directly into the hook and loop strap. Close the strap around the transducer.

4. Legal Notices

The manufacturer Novo Klinik-Service GmbH accepts no liability for damages (in particular for functional failures, injuries, infections and/or other complications or other undesirable incidents) caused by unauthorized product modifications or improper handling, care (cleansing/disinfection) and/or storage of the products against the provisions of this instruction for use. Where legally permissible this is valid for resulting damages to the products themselves as well as for all resulting consequential damages. Product modifications on the part of the manufacturer are reserved at any time.

5. Ordering information

REF	Product	Width	Length	SU
30-45	Transducer-Fix H	12 cm	45 cm	36
30-45-03	Transducer-Fix V	12 cm	45 cm	36

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