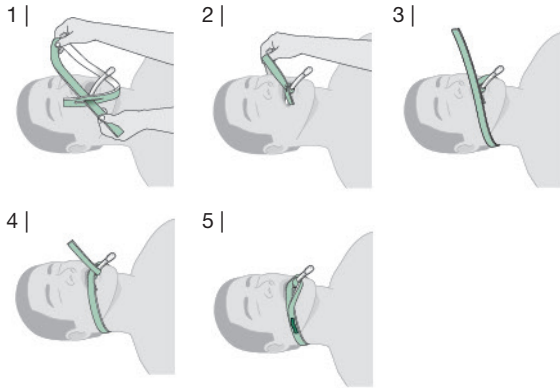
TuBo-Clip

Fixierband für Endotrachealtuben | Endotracheal Tube Holder

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1. Product Description/Application Area

Holder for the safe fixation of endotracheal tubes (oral/nasal), laryngeal tubes and laryngeal masks.

2. Warning Notices

Tube holders have to be regularly reviewed on their correct positioning.

3. Application with connected tube

1 | Form a loop around the tube by conducting the end of the tube holder through the slot. The white material is directed at the tube.

2 | Tighten the loop around the tube.

3 | Conduct the tube holder around the patient's neck.

4 | Wind the tube holder under slight tension one to two times around the tube. The white material is directed at the tube.

5 | Close the ends under tension with the hook and loop strip. If necessary, cut the excess part off with scissors.

It is possible to additionally fix a bite guard if required.

4. Replacement Frequency

We recommend to replace the tube holder after 24 hours at the latest.

5. Legal Notices

The manufacturer Novo Klinik-Service GmbH accepts no liability for damages (in particular for functional failures, injuries, infections and/or other complications or other undesirable incidents) caused by unauthorized product modifications or improper handling, care (cleansing/disinfection) and/or storage of the products against the provisions of this instruction for use. Where legally permissible this is valid for resulting damages to the products themselves as well as for all resulting consequential damages. Product modifications on the part of the manufacturer are reserved at any time.

6. Ordering information

REF	Product	Product Description	Length	SU
20-01	TuBo-Clip	Foam strap with hook and loop strip	72 cm	100
20-05	TuBo-Clip XL	Foam strap with hook and loop strip	85 cm	100

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