


GEBRAUCHSANWEISUNG | INSTRUCTION FOR USE

Novo-Pad V | Novo-KidsPad V

Tracheal-Vlieskomresse | Tracheostomy Non-woven Dressing

Novo-Pad V

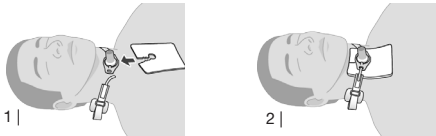

Novo-KidsPad V



1 | Legen Sie die Komresse mit der beschichteten, glatten Materialseite zur Haut gerichtet um die Kanüle.

1 | Place the dressing with the coated and smooth material side against the skin and around the cannula.

2 | Fertig angelegte Komresse.

2 | Dressing in final position.

MATERIAL | MATERIAL








Polyethylen-Beschichtung | Polyethylene coating
Vliesmaterial | Non-woven material

ACHTUNG | CAUTION

Bei Hautirritationen oder allergischen Reaktionen muss die Trachealkomresse sofort entfernt und ein Arzt informiert werden. Stellen Sie sicher, dass während des Wechsels der Trachealkomresse die Trachealkanüle gesichert und/oder eine ausreichende Sauerstoffzufuhr gewährleistet ist.

In case of skin irritations or allergic reactions, the tracheal dressing must be removed immediately and a doctor should be informed. Ensure that the tracheal cannula is secured and/or sufficient oxygen supply is guaranteed when replacing the tracheal dressing.

PIKTOGRAMM-LEGENDE | PICTOGRAM LEGEND

-  Chargenbezeichnung | Batch code
-  Gebrauchsanweisung beachten | Consult instruction for use
-  Herstellungsdatum | Date of manufacture
-  Nicht wiederverwenden | Do not reuse
-  Hersteller | Manufacturer
-  Von Sonnenlicht fernhalten | Keep away from sunlight
-  Trocken aufbewahren | Keep dry
-  CE-Kennzeichnung | CE marking

REF	BESCHREIBUNG DESCRIPTION	BREITE WIDTH	LÄNGE LENGTH
1 70-11	Novo-Pad V 1-lagig, beschichtet, unsteril 1-layer, coated, non-sterile	8,2 cm	9,5 cm
2 70-12	Novo-Pad V 2-lagig, beschichtet, unsteril 2-layer, coated, non-sterile	8,2 cm	9,5 cm
3 70-21	Novo-KidsPad V 1-lagig, beschichtet, unsteril 1-layer, coated, non-sterile	6,5 cm	6,5 cm
4 70-22	Novo-KidsPad V 2-lagig, beschichtet, unsteril 2-layer, coated, non-sterile	6,5 cm	6,5 cm