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
en

en

1. Product Description/Application Area

BePa-KidsClip® VARIO is a tracheostomy tube holder for the safe fixation of a tracheal cannula. It is applied in neonates, infants and children.

2. Indications

The tube holder is used to safely fix a tracheal cannula.

3. Contraindications

There are no known contraindications.

4. Patients Targeted

Tracheostomised patients

5. Intended Users

Doctors, nursing staff and patients/nursing relatives

6. Application Description

Size selection:

Please measure the neck circumference of the patient (from one cannula plate to the other) and select an appropriate padding length. Make sure that there is a sufficient distance maintained between the holder and the neck of the patient.

1 | When applying the tracheostomy tube holder around the neck, the hook or hook and loop fasteners are pointing outward. Insert the hooks or the hook and loop into the lateral eyelets of the tracheostomy plate and fix the hooks or the hook and loop.

2 | If necessary, adjust the length of the holder by displacing the sliders on the elastic band.

7. Useful Life:

The useful life of the product depends on the individual use and handling. The product has to be replaced immediately if damage occurs. For hygienic reasons in the clinic, we recommend to change the holder after 24 hours at the latest.

8. LOT Number

The LOT number is imprinted on the product and outer packaging.

9. UDI Carrier

The UDI-Code is imprinted on the product and outer packaging.

10. Expiry Date

The product is not subject to any expiry date.

11. Storage Conditions

Keep BePa-KidsClip® VARIO dry and away from sunlight.

12. Handling Conditions

In case of damaged packaging or inappropriate storage the product may not be used and has to be destroyed.

13. Sterile Condition

The product is non-sterile.

14. Sterilisation Process

Not applicable

15. Sterile Packaging

Not applicable

16. Warning Notices

The tracheostomy tube holder may not be applied too tight on the neck (risk of strangulation). Make sure that there is an optimal distance of about 1 cm between the holder and the neck of the patient. To prevent the tracheal cannula from becoming unintentionally loose, it is necessary to check the safe hold of the tube holder regularly. There is a risk of pressure marks (decubiti) and skin lesions when the application is incorrect or too tight. Failure to comply with the instructions for use can cause a dislocation of the tracheal cannula.

17. Single Use Notice resp. Notice on Reuse

The product is a disposable product. Do not reuse. The reuse of disposable products holds a potential risk for patients and users. This can result in contamination and/or effects on the functioning. Contamination and/or limited functioning can cause injuries, diseases and the death of a patient.

18. Reprocessing Notice

The product may not be reprocessed.

19. Notice on Processing Methods

Not applicable

20. Disposal

The product can be disposed with normal clinical waste or according to local regulations. Special guidelines of medical facilities for the disposal (of e.g. multi-resistant pathogens MRSA, VRE) need to be taken into consideration.

21. Notice on the Application with other Products

BePa-KidsClip® VARIO is compatible with all standard tracheal cannulae.

22. Notice on Medical Devices/Custom-made Products/Products for Clinical Testings

Medical device of Class I. Accessory product for tracheal cannulae.

23. Notice on the Application by Laymen

BePa-KidsClip® VARIO is easy to apply and does not require special abilities or equipment. In case of unexpected complications (such as allergic reactions) immediately inform a doctor.

24. Notice on Reporting Requirements

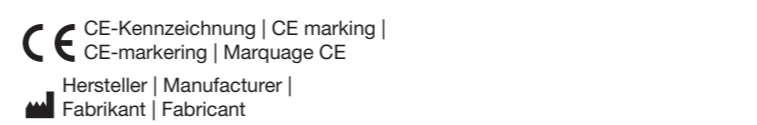
Please inform the manufacturer and the responsible authority of the member state in all cases of serious incidents in connection with the product.

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